

FOUNTAIN

[INSTRUCTION FOR USE]

BALLOON INFLATION DEVICE

SCOPE OF APPLICATION

The products are mainly used in PTCA operation, the balloon dilatation catheter is pressurized in order to achieve the expansion of blood vessels or stay in the purpose of intravascular stent.

CONTRA-INDICATIONS

None known.

THE USE OF THE ENVIRONMENT AND STORAGE

This product should be stored in a cool dry environment, avoid direct sunlight.

PLEASE PAY ATTENTION TO THE FOLLOWING POINTS

1. It must be operated by trained doctor.
2. This product is sterilized by ethylene oxide, if damaged packaging use is prohibited.
3. This product do not use in the process of arbitrarily change its structure.

THE PRODUCT STRUCTURE AND MATERIALS

Balloon inflation device is composed of an outer sleeve, a core pipe, a pressure gauge, a connecting tube and a conical joint, etc.
The outer sleeve and the cone joint is made of PC made; core pipe ABS connecting tube PU; pressure gauge is mainly composed of copper, iron, and PC.

SPECIFICATIONS

Balloon inflation devices has normal type and gun type, with specification of BID1, BID2, BID3, BID4, BID5, BID6, BID7, BID8, BID9, BID10, BID11, BID12, BID13, BID14, BID15, BID16, BID17, BID18, BID19, BID20, BID22, BID23, BID24, BID25, BID31, BID32, BID33, BID34, BID35, and BID36.
Volume: 20ml, 25ml, 30ml.

INSTRUCTIONS FOR USE

1. Remove the packaging, if damaged, then prohibition of use.
2. The balloon inflation device is respectively connected to the balloon catheter joint, push the core rod to give the pressure to the balloon, in the meanwhile, check if the pump is sealed well and pump pressure table works properly and the balloon is complete or not.
3. After confirmation of the above, then under the X ray developing, slowly push the contrast agent to the balloon catheter to implementation of angiography.

WARNING

1. This product is valid for three years, Please confirm the sterilization date before use.
2. If the pressure pump extravasation occurred, leakage and so on, then prohibition of use.
3. To pressure if the pressure gauge pointer does not move, it is prohibited to use.
4. The pressure applied by the pressure pump can not exceed the reading of the pressure gauge.
5. When it is pressurized to the balloon, the gas is prohibited as medium.
6. The product is single use only, prohibition of re-sterilization and re-use.



Do not reuse



Sterilization using ethylene oxide



Attention, see instruction for use



Fragile



Keep away from sunlight



Consult instructions for use



Do not re-sterilize



Store in cool place



Keep dry



damage to packing do not use



Use by



Lot Number



Catalog Number

SHUNMEI MEDICAL CO.,LTD.

Add: R401 of building B, No.8 of 1st Jinglong Road,
Baolong Industrial Zone, LongGang District
518116 Shenzhen, Guangdong, China.

Tel: 0752-3306949

Email: nora@shunmed.com

Web: www.shunmed.com

Lotus NL B.V.

Add: Koningin Julianaplein 10,
1e Verd, 2595AA, The Hague, Netherlands.

Tel: +31644168999

Email: peter@lotusnl.com