

【INSTRUCTION FOR USE】

ARMOR

HEMOSTASIS VALVE SET

SCOPE OF APPLICATION

Interventional effect on intravascular disposal and check the guide wire or as the diagnosis instrument

CONTRA-INDICATIONS

None known.

THE USE OF THE ENVIRONMENT AND STORAGE

This product should be stored in a cool dry environment, avoid direct sunlight.

PLEASE PAY ATTENTION TO THE FOLLOWING POINTS

- 1, it must be operated by trained doctor.
- 2, this product is sterilized by ethylene oxide, if damaged packaging use is strictly prohibited.
- 3, this product for one-time use, do not repeat the sterilization repeated use.
- 4, this product do not use in the process of arbitrarily change its structure.

THE PRODUCT STRUCTURE AND MATERIALS

This product is composed by the needle, torque device, Y type connector and the extension tube.

needle made of polycarbonate needle base and stainless steel needle tube, a needle tube length 90.0-150.0mm, a needle tube diameter 0.80-1.10mm. Torque device is made of brass, chuck and polypropylene the front end of the handle, the front end of the long resin composition, 15.0-20.0mm, chuck long 11.0-13.0mm.

Type Y connector by a dial, the main branch and side holes, rotary switching terminal, hemostasis valve and fixed valve

INSTRUCTIONS FOR USE

- 1,remove the packaging, if damaged, then prohibition of use.
- 2, Y type connector is connected with the catheter, and the guide wire inserted into the needle, then through Torque device.
- 3,the guide wire insertion the needle type Y connector, push the torque device, in vitro help the guide wire slowly into the body.

WARNING

- 1 the duration of the sterilization of products for three years, please confirm the sterilization date before use, prohibited the use of expired.
- 2 this product is a disposable product, it is strictly prohibited to repeated sterilization, repeated use.

SHUNMEI MEDICAL CO.,LTD.

Add: R401 of building B, No.8 of 1st Jinlong Road,
Baolong Industrial Zone, LongGang District
518116 Shenzhen, Guangdong , China.

Tel:0752-3306949

Email:nora@shunmed.com

Web: www.shunmed.com

Lotus NL B.V.

Add: Koningin Julianaplein 10,1e Verd,
2595AA, The Hague, Netherlands.

Tel: +31644168999

Email: peter@lotusnl.com



Do not reuse

STERILE EO

Sterilization using ethylene oxide



Attention, see instruction for use



Fragile



Keep away from sunlight



Consult instructions for use



Do not resterilize



Store in cool place



Keep dry



damage to packing do not use



Use by

LOT

Lot Number

REF

Catalog Number