

GUIDER

【INSTRUCTION FOR USE】

INTRODUCER SET

Intended use:

The introducer set is single use device allowing for introduction, manipulation, and removal of stimulation leads after percutaneous entry is gained with a needle.

Contraindications:

Physician judgment should be used in determining for which patients percutaneous interventional procedures are contraindicated.

Warning:

- 1.This product is antiseptic by Ethylene oxide. If the packing material has breakage, don't use.
- 2.This product is only supplied for disposable use, should not be reused.
- 3.This product shall not be changed its structure in the use process.
- 4.The product must only be used by qualified personnel who are familiar with the technique.

Instruction for use:

1. Perform a skin incision under local anesthesia.
2. Do a needle puncture using ultrasound or X-ray and aspirate with the attached syringe.
3. After successful puncture, advance the guidewire through introducer needle .
4. Remove the needle and dilate with the fully assembled coaxial sheath/dilator.
5. Loosen the dilator, checking that the tip of the outer sheath has been properly placed inside the cavity .
6. Remove the dilator and guidewire.

Storage:

Store in a cool, dry area.

Caution :Due to the risk of exposure to HIV(human Immunodeficiency Virus)or other blood borne pathogens .Healthcare workers should routinely use "universal blood and body fluid precautions" in the care of all patients .

SHUNMEI MEDICAL CO.,LTD.

Add: R401 of building B, No.8 of 1st Jinglong Road, Baolong Industrial Zone, LongGang District 518116 Shenzhen, Guangdong , China.

Tel: 0752-3306949

Email: nora@shunmed.com

Web: www.shunmed.com

Lotus NL B. V.

Add : Koningin Julianaplein 10, 1e Verd, 2595AA, The Hague, Netherlands.

Tel : +31644168999

Email : peter@lotusnl.com



Do not reuse

STERILE **EO**

Sterilization using ethylene oxide



Attention, see instruction for use



Fragile



Keep away from sunlight



Consult instructions for use



Do not re sterilize



Store in cool place



Keep dry



damage to packing do not use



Use by

LOT

Lot Number

REF

Catalog Number